

RT40 Buttons



Getting Started

Turn the RT40 on using the red button at the bottom right hand corner of the machine. You may need to press for a couple of seconds until you feel the machine vibrate.

When it is first started or after you wake it up you will see the following screen:




Swipe this screen up towards the top using your finger or thumb and you will see:

In the centre of this screen you can see the Comtag icon. Tap on this icon to start the TB Testing programme




You will then see:

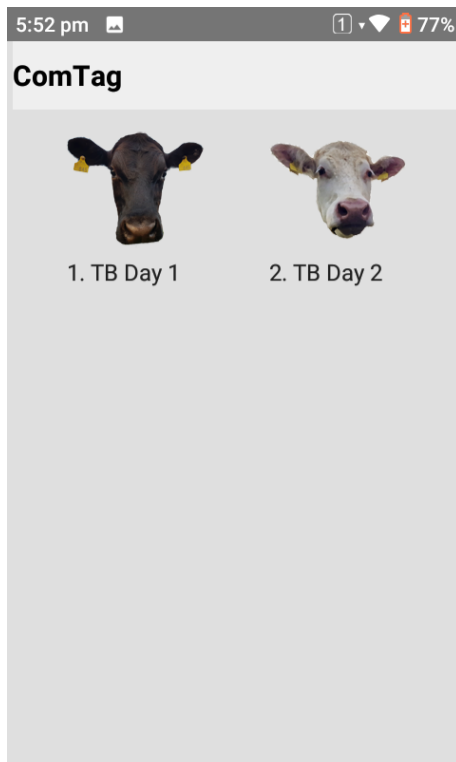

From this screen you can either tap on the icon for Day 1 or Day 2 or press 1 or 2 on your keypad to start whichever you want.

Day 1 Programme

On choosing Day 1 you will see a screen showing you the list of tests available to you with the most recent at the top of the list:

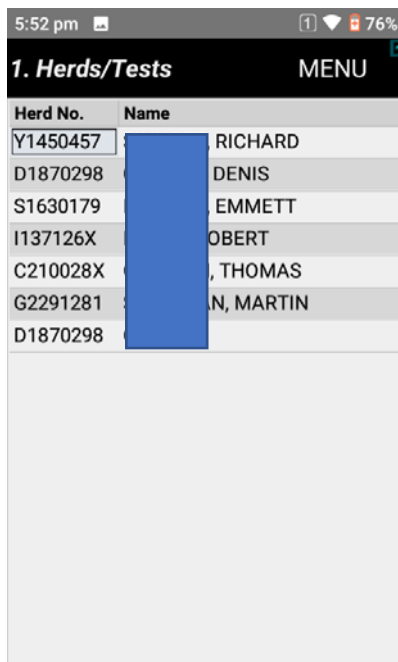

You can select the test you want by tapping the screen or by pressing the arrow keys on your keypad. Once the test you want is highlighted press Enter to choose it and you will then be moved on to the Farm Details screen

Press Enter again (or F1) and you will see the Test Details screen:

2:13 PM

3. Test Details MENU

Test **TB**

Interpretation **Standard**

TB Day1 **21/11/22**

TB Test **WHT**

Whole Herd Test

Tuberculin A

B

5:53 pm 76%

2. Herd Details MENU

Herd No. **Y1450457**

Country

Herd ID

Owner Name

IARD

Address

Comment

File **04573926**

On this screen you can edit the type of Test (Just TB or TB and Blood Testing), the date of the test, the TB Test Reason, the Tuberculin Batch numbers and the Interpretation if this is necessary.

Press the Enter key or F1 and you will proceed into the Search Screen where you can start searching for the animal that you are testing.

Search Screen

5:53 pm 76%

4. Test - Find Tag MENU

Find Tag

B...	Tag Number	Bre
A	372 2251875 5 0141	CH
	372 2177245 4 0297	BE
	372 2164544 2 0457	LM
	372 2121977 1 1328	LM
	372 2193584 4 2554	FF
	372 2193584 9 2567	FF
	372 2180486 2 1080	FF
	372 2144399 5 1365	FF

This search screen allows you to find the animal that you are testing but it also gives you information about the animals and if they have already been tested you will see that they are coloured green.

As you type in the Find Tag box the list will be filtered to show the animals that match your selection. If you enter 7 it will show you all of the animals whose number ends in 7. If you enter 37 it will show you all of the animals that end in 37 and so on.

In this case there is only one animal ending in 37 so if you now press Enter you will be taken directly to that animal so that you can enter their skin readings.

2:32 PM

4. Test - Find Tag MENU

Find Tag

1 match

B...	Tag Number	Bre
	UK362513 500537	CH

2:32 PM

4. Test Animals MENU

UK362513 500537

CHX Cow 2/9

Avian

Bovine

Blood

Clinical

Not Tested

If this animal's skin readings were 6 over 6 you press 6 then press the Enter key twice. The skin readings will be entered and you will be returned to the Search Screen. If the skin readings were 6 over 8 you press 6 followed by Enter and then 8 followed by Enter. On every animal other than the first you will then be taken back to the Search screen to find the next animal.

When you are doing the first animal in the test you will be prompted for a name for Batch A. By default this is set to say Herd. If you are planning to split the test into batches you can alter this to give it whatever name you want e.g. Milkers.

Entering text on the RT40 is really easy. On the left-hand side of the RT40 there are two buttons. The lower button (with an orange outline) is the wakeup button which you can use if the machine goes into sleep mode. The top button controls the on screen keyboard. Press it and a keyboard will appear on the screen which you can use to type in whatever you want. When you've finished typing press the button again and the keyboard will disappear.

Alternatively the red button at the bottom left of the machine can be used to change the actual keyboard into a letter keyboard rather than numeric. You can use either method but if you do press the red button remember to press it again when you finish typing to change it back to numeric.

2:16 PM

4. Test Animals MENU

UK381935 502967

SD Heifer 1/6

Avian 7

Bovine 7

Blood A

Barn2

Clinical Lump

Not Tested

Clinical Remarks

Once you have entered the skin readings you will be taken back to the Search screen. If you want to enter a Clinical Remark for the animal that you have just entered the skin readings for press Esc or Back button and you will be taken back to that animal. Tap the Clinical Remark box on the Animal screen and you can then type in your remark. Once you have entered your remark it will be high lighted in red so that it is easy to spot when you go back to that animal on the reading day. To get back to the Search screen press the Search button – the large button with an orange outline just below the screen.

2:14 PM

4. Test Animals MENU

UK381935 602996

SD Heifer 1/6

Avian 7

Batch D Description

Cancel OK

Changing Batch


If you want to change batches press the F5 button (or press Menu and choose New Batch from the drop down menu). You will then be asked whether you want to and Batch A and start Batch B(in this case C to D!). Select yes and you will see a screen that allows you to enter a name for the new batch. Type in the name you want to use and tap OK or press Enter.

End of the Test

When you have finished the test you will find that any animals that have day 1 readings will be green and animals that haven't been tested will be grey. Once you are happy that you have entered all of the day readings you can escape back to the list of tests if you have another test to read or you can exit the programme by pressing Menu and the 0 to End Comtag and then tap Yes on the screen or arrow across to the Yes button and pressing Enter.

Day 2

Choose Day 2 on the initial Comtag screen to get into the Day 2 programme. The initial screens will be the same up to and including the Search screen. Most of the animals on the Search screen will now be coloured Green if they were tested on Day1.

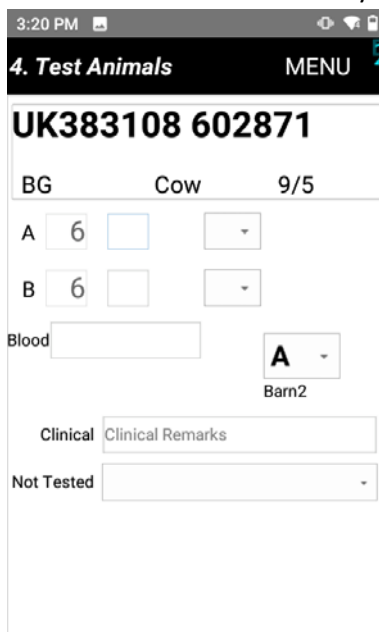

3:20 PM

4. Test Animals MENU

UK383108 602871

BG Cow 9/5

A 6

B 6

Blood


A

Barn2

Clinical Clinical Remarks

Not Tested

Use the Search screen in the same way as on Day 1 to find the animal you wish to record. Once you are in the animal screen if there are no changes to the skin readings press the Enter key twice and the same readings will be entered for Day 2 as you entered on Day 1 and you will be taken back to the Search screen.

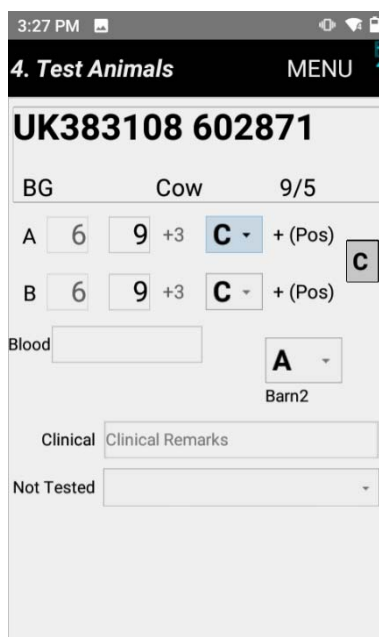

3:27 PM

4. Test Animals MENU

UK383108 602871

BG Cow 9/5

A 6 9 +3 **C** + (Pos) **C**

B 6 9 +3 **C** + (Pos)

Blood


A

Barn2

Clinical Clinical Remarks

Not Tested



If there are changes to record enter the new readings by pressing the number followed by the Enter key. You will now be taken to the Reaction boxes. By default if there is an increase in skin thickness the reaction box will be set to C for circumscribed.

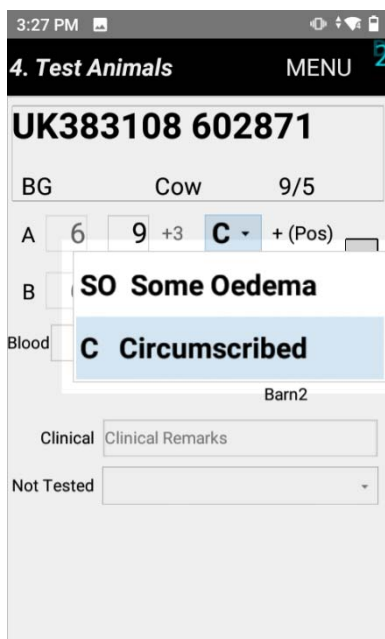

If you wish to record Oedema tap the relevant box or press 0 to open a small dropdown menu from which you can record Some Oedema. You can use the up arrow key to move to Some Oedema and press Enter or tap it on the screen (I would recommend the arrow key). Once you have recorded the oedema or if you are happy with Circumscribed press Search button to take you back to the Search screen.

If your skin readings and reaction description give an Inconclusive result you will hear a bleeping sound and the Result box will turn amber and contain an I.

If your skin readings and reaction description are sufficient to create a Reactor you will hear a bleeping sound and you will be prompted to enter a Reactor tag. Once this is entered you will see that the Result box has turned red and contains an R. When you return to the Search screen you will see that this animal is coloured red.

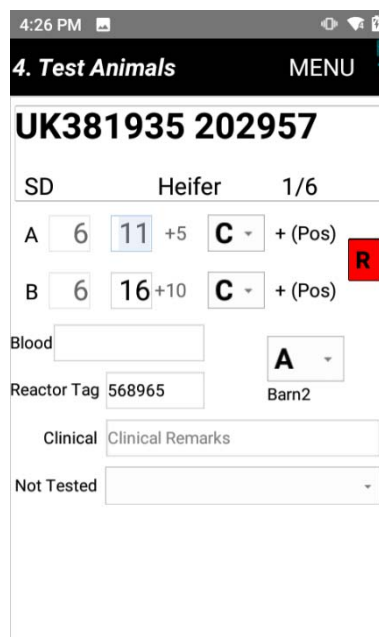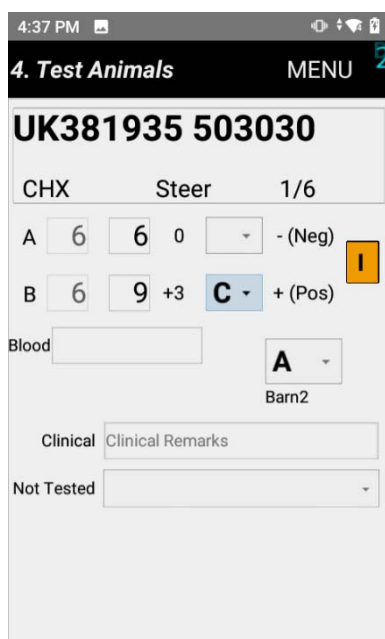


End of the Test

When you have finished the test you will find that any animals that have had readings entered for both days will be either Blue or Red. Animals that have only got day 1 readings will still be green and animals that haven't been tested at all will be grey. Once you are happy that you have entered all of the day 2 readings you can escape back to the list of tests if you have another test to read or you can exit the programme by pressing Menu and the 0 to End Comtag and then tap Yes on the screen or arrow across to the Yes button and pressing Enter.

Transferring Files

Transferring files to and from the RT40 is simple. Using the cable provided attach the RT40 to your PC/laptop. The following screen will appear

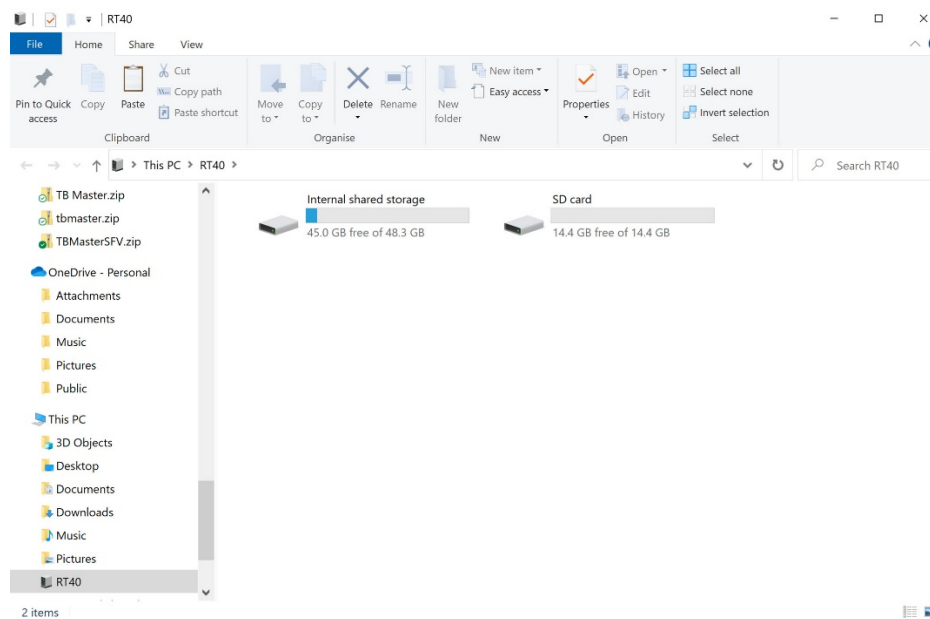

If it doesn't appear disconnect it and reconnect it – it sometimes needs a second look! Once this screen has appeared minimise it and then transfer the test to or from TBMaster as per usual.

Comtag TB File Updates

If there is an update available for your TB software you will be sent a file via e-mail or a link to that file. Download it and save it somewhere easy to find. Right click on the file and select Copy.

Connect your machine as shown above

Double click on the Internal Shared Storage and you will see a list of folders appear.

Double click on the Download folder

A list of files will appear (there may be only one!). Right click somewhere on the window where there isn't a file and select Paste. Alternatively you can hold the CTRL button and press V.

If a window appears saying that the file already exists choose the top option which says Copy and Replace.

Once you have done this on your handheld make sure that the testing programme is closed and return to the home screen (pressing F9 takes you to the Home)

Tap on the Files icon to open the Files App.

At the top of your Recent Files you will see ComtagTB_GB.apk

Tap on this file and you will see a message concerning protecting your personal data – press Continue

Now press Install

Once installation is complete you will see a screen saying Open or Done. Tap on whichever you wish.